


Express with Safari - new performance updates

In the next few weeks Express iPads will be updated to provide better performance, and as part of this upgrade you'll notice that your iPad screen has changed slightly. The change will be your indicator that the update is complete. Once done, you'll simply need to drag the Express and Express Info icons back to the Dock Bar and move all other icons to your utility folder.

Currently your Express iPads appear configured in the following way.

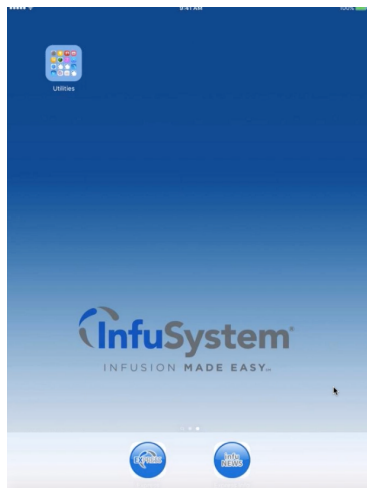

Once updated they will appear as either one below:



To move the Express and Express Info icons back to the Doc bar simply:

1. Hold your finger on the Express icon until all icons “jiggle”
2. Drag Express and then Express Info to the Dock
3. If Safari is on the Dock Bar, move it to Utilities <Note: The Safari icon in this upgrade, will take the user directly to Express.>
4. Press the Home button on the lower portion of the iPad screen (actual button) to stop icons from “jiggling” and lock them in place