



## InfuBLOCK™ Pump Return

Dear Patient,

Thank you for trusting your pain management care to InfuSystem. As the final step in your care, you are instructed to return the pain management pump and black pump pouch to InfuSystem. We've made this step easy for you. All you need is the prepaid return shipping box provided to you at discharge.

Please carefully follow these instructions.



At the end of your infusion, press the power button to turn off the pump, then press "Yes" to power down. This will shut off your pump.



Securely place the ziplock bag with the pump and black pump pouch into the prepaid return shipping box.



Remove the extra batteries from the return box.



Seal the box using the stickers provided.



Remove the medication bag from the black pump pouch.



Return the box via the U.S. Postal Service. You can place the shipping box in your mailbox, hand it to your mail carrier or drop it off at your local post office.



Cut the tubing on the right and left sides of the pump and discard it along with the catheter and medication bag.



Postage is prepaid. No additional postage is required!



Place the pump in the plastic ziplock bag provided (some remaining tubing will be attached).

**Pump Return Questions? Call the InfuSystem 24/7 Hotline at 1-844-724-6123.**



**The box is ready to ship. DO NOT remove the prepaid shipping label already affixed to the box.**